



QUICK START GUIDE SENTIERO DESKTOP SCREENER (BLUE HOUSING)

DEVICE CONNECTORS

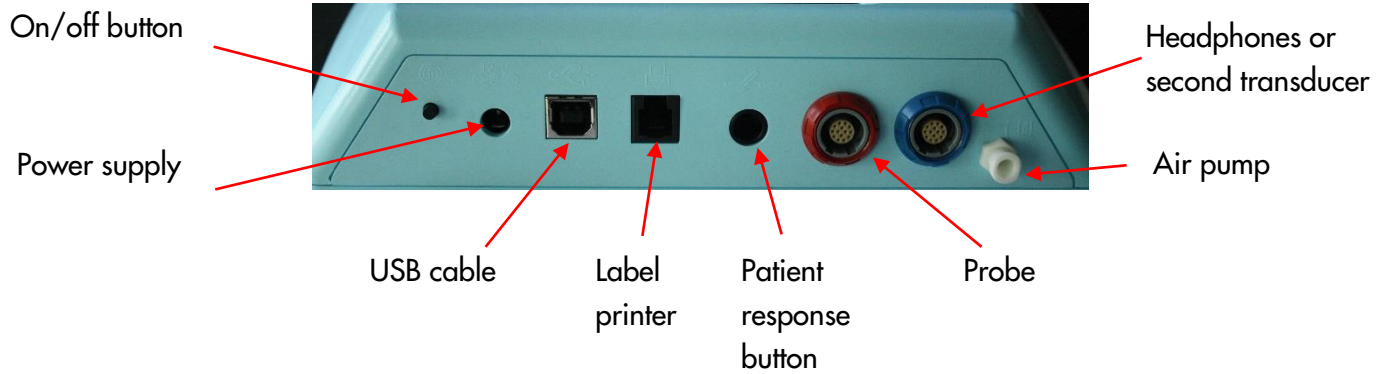

START MENU

Please note that not all Start Menu options will be available depending on the configuration of your device

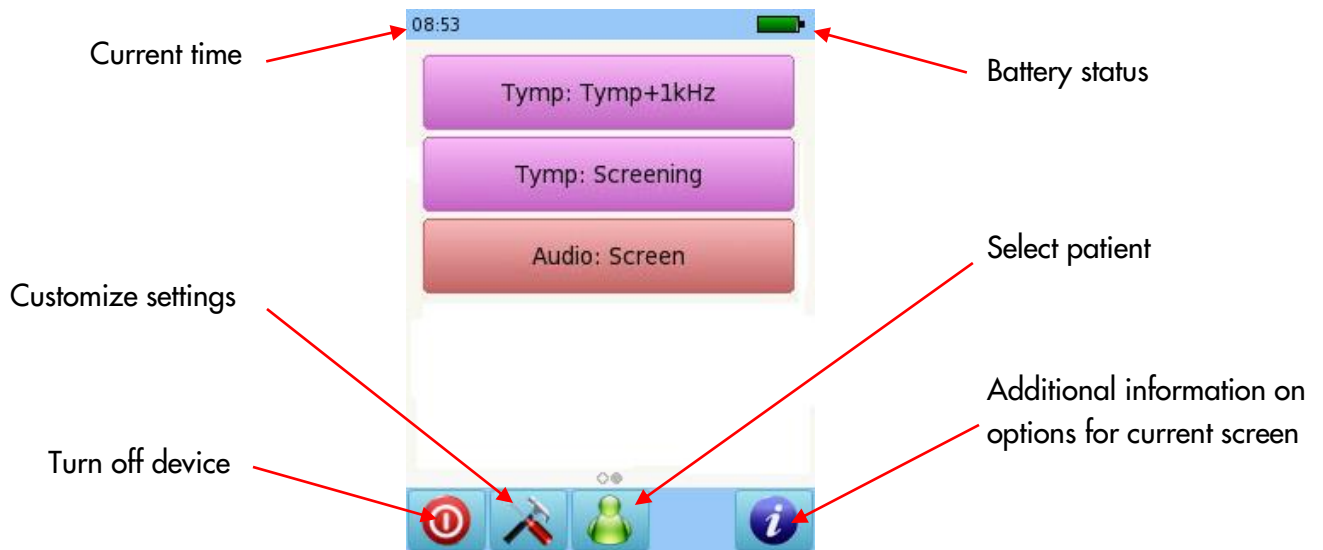


QUICK START GUIDE SENTIERO DESKTOP SCREENER (BLUE HOUSING)

TESTING

TYMPANOMETRY



Select the Tymp: Tymp +1 kHz button from the start menu and connect the probe to the red connector and the air cable to the pump connector. Select the test ear by pressing either the Right or Left ear icon. Cartoon mode will show an airplane on the screen during the tympanometry test. For web descriptions SOD02 and SOD03, connect a second transducer to the blue connector for Contralateral Reflex testing.

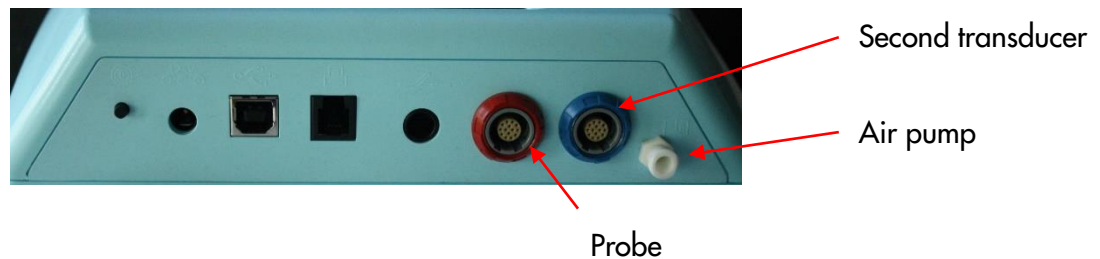

Select the appropriate ear tip size and place it on the probe tip. Insert the probe in the ear making sure that the ear tip fits tightly into the ear canal.


TYMPANOMETRY TEST SCREEN

The system will first check the probe fit and wait for the pressure build up. The LED light on the probe will pulse at a steady pace to indicate a correct fit. During the test, if the probe fit is not correct, the LED light will blink at a fast pace and an error message prompting you to retry or cancel the test will appear on the screen.

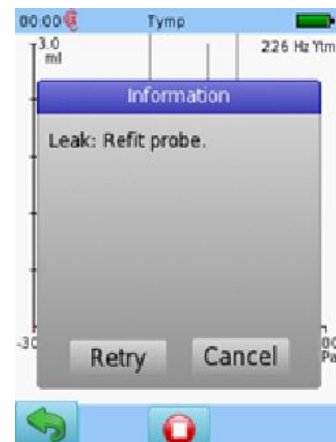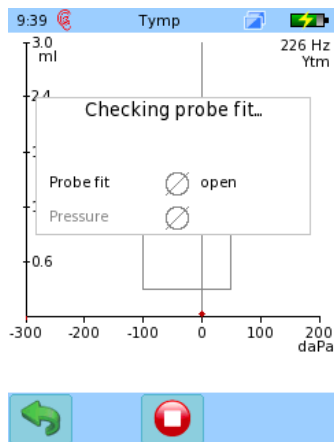


QUICK START GUIDE SENTIERO DESKTOP SCREENER (BLUE HOUSING)


Once a proper seal has been obtained, the test will proceed. A tympanometry test will be performed, followed by a reflex measurement (when Tym + 1kHz selected).

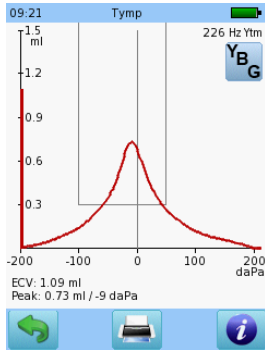

Tympanometry test result

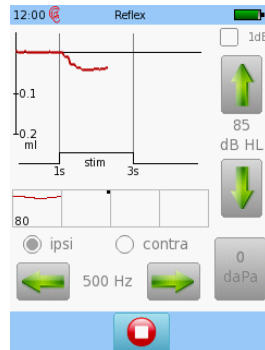

Reflex test in progress

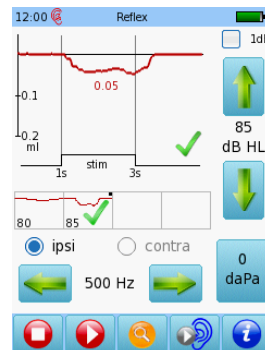

Reflex threshold found



Overall Reflex test result

AUDIOMETRY (Available only on web description SOD02)

Select the Audiometry: Screening button from the start menu and connect the headphones to the blue connector. Select the ear on which you want to start the test.



QUICK START GUIDE SENTIERO DESKTOP SCREENER (BLUE HOUSING)

AUDIOMETRY TEST SCREEN

Stimulus on indicator

Select the frequency

Stop test

Select the ear to test

Select the stimulus levels

Press to present the stimulus

Additional information on options for current screen

Select the type of stimulus: pure tone — pulsed tone ... or warble FM

As you present the stimulus, the current level will be displayed on the corresponding square. Once the hearing status for a frequency has been identified, move on to the next frequency.


Tapping twice on a square will display NR (no response).

Tap three times to get back to blank square.

Once all frequencies have been tested for one ear, you can test the other ear.

QUICK START GUIDE SENTIERO DESKTOP SCREENER (BLUE HOUSING)

When the test is finished, press the Stop button. You can add the patient's personal data, and save on the device, print to a label (a label printer is required) or print to a pdf file or full page report on PC.

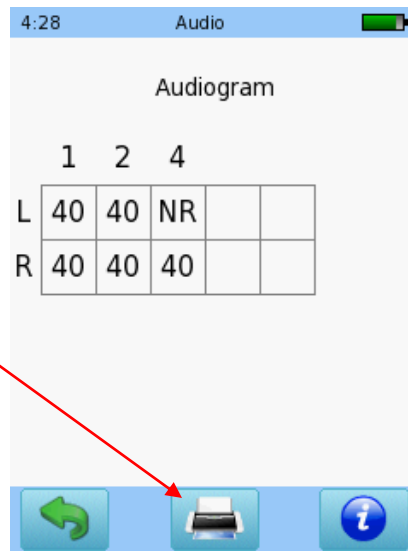

QUICK START GUIDE

SENTIERO DESKTOP SCREENER

(BLUE HOUSING)

Quick Guide Information

Release date: 2019-04
Revision: 03
Valid from: Firmware Rev. 2.5, Mira PC Software Rev. 2.0

Applies to: Model: SOD100497, web description SOD01, SOD02 and SOD03.

All mentioned items, products, brands and trademarks are registered or owned by the mentioned companies.

All information, illustrations, and specifications provided within this manual are based on the latest product information available at the time of publication. PATH medical reserves the right to make changes at any time without notice.

Please note that not all products displayed on our webpage or advertising materials are available in all territories. Please contact your local distributor or PATH medical for further information.

The latest revision of the Quick Start Guide is available online at www.pathme.de/download.

Errors and omissions excepted.

Copyright Notice

No part of this manual may be reproduced, translated, stored, or transmitted, in any form or by any means, electronic, mechanical, photocopying, recording, or otherwise, without the prior written consent of PATH medical GmbH.

Copyright © 2019 PATH medical GmbH

PATH medical GmbH
Landsberger Straße 65
82110 Germering
Germany

Tel.: +49 89 800 76 502 Fax: +49 89 800 76 503 Internet: www.pathme.de

CE
0124

